

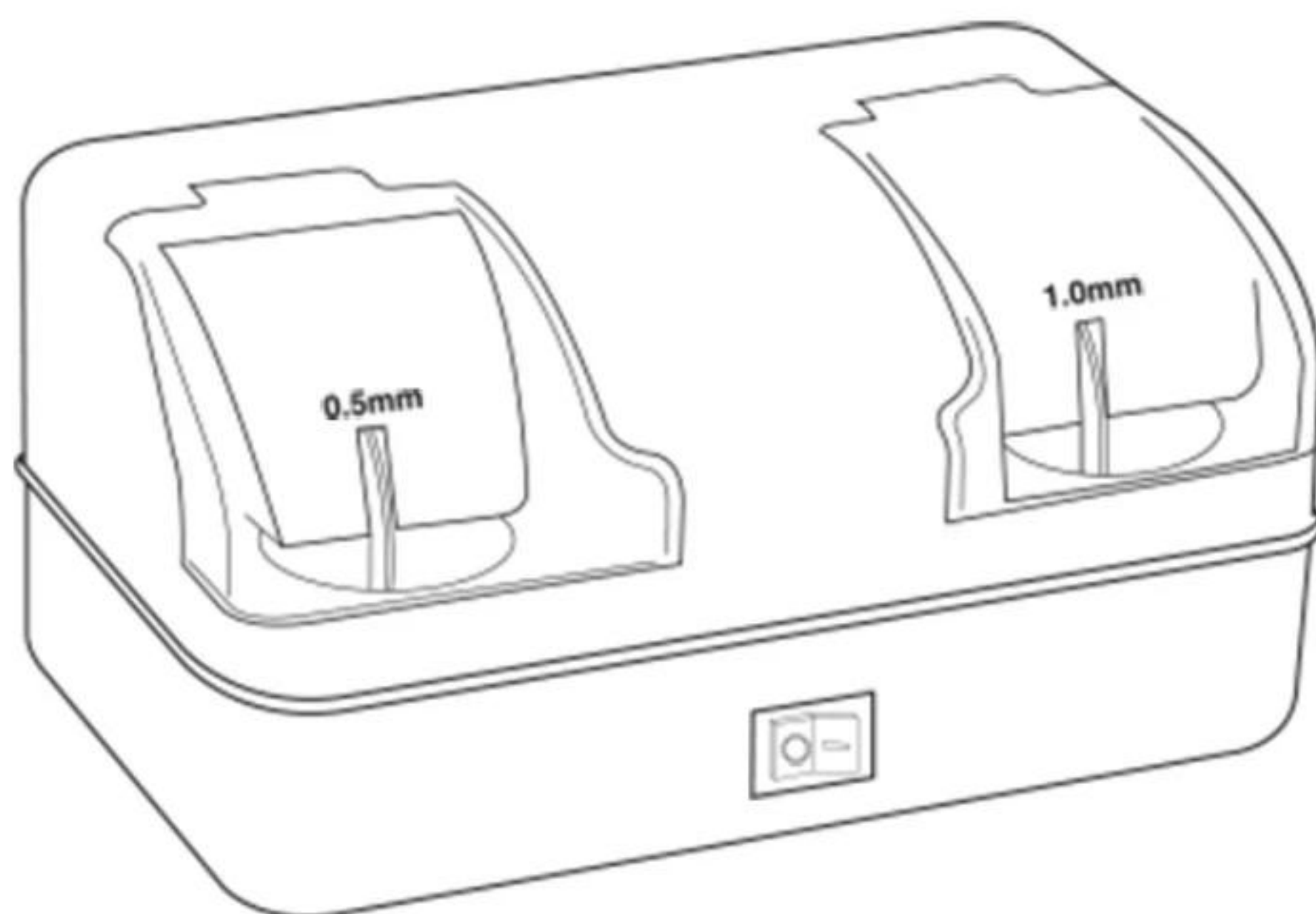


AN-1000 NOTCH-CUTTING MACHINE

User's Manual

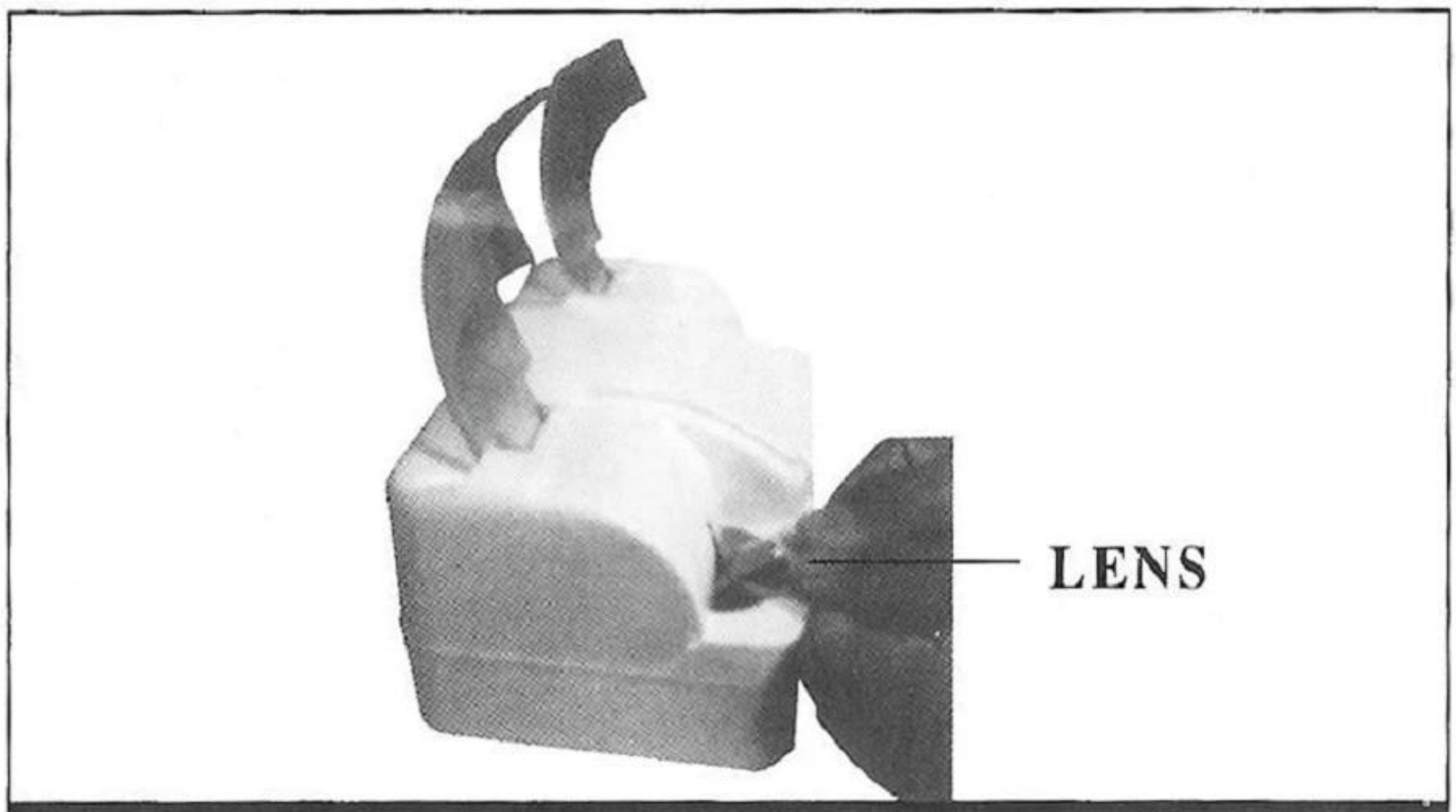

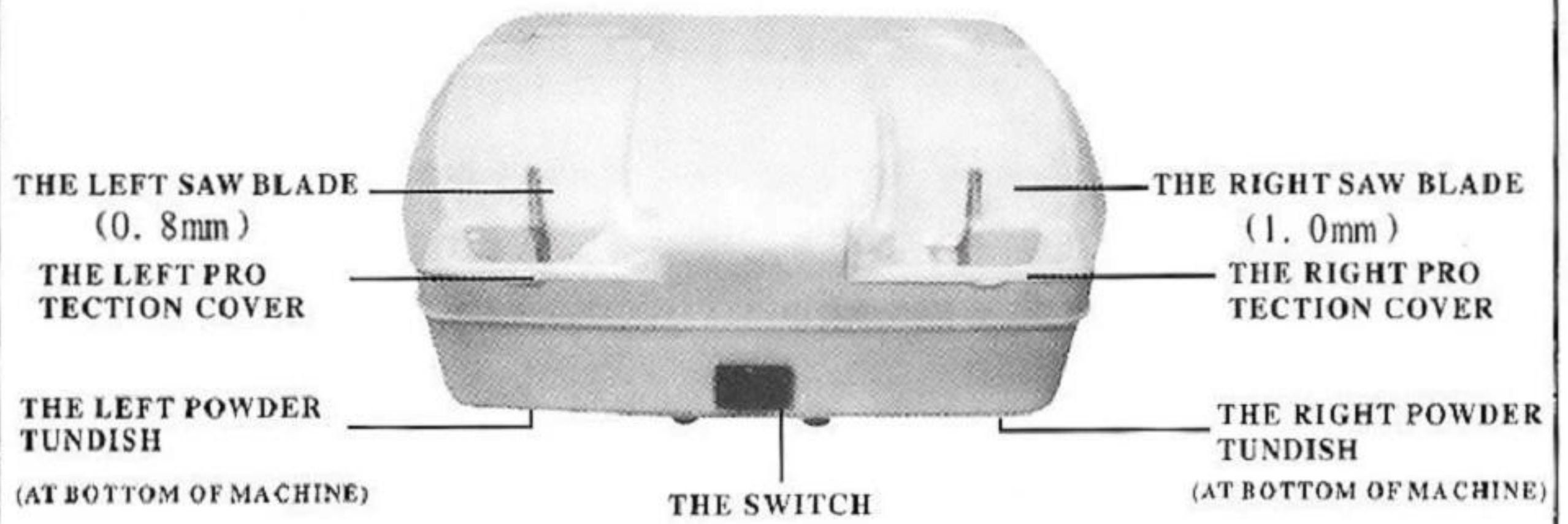

APPLICATION DRAWING

Note: should clear up powder once every month

INFORM USERS

Don't operate the machine before you have not read over and totally got a thorough understanding of all rules. Not observing the manual and stipulating that will cause fire, getting an electric shock or such accidents as individual damage. For the consistent safety in operation and instructing others, please read manuals more.

Note: To the accident which the operator causes because of being careless or ignore the rule of preventing, that the maker does not shoulder any responsibility.

WARNING: THIS PRODUCT FORBIDS MINORS TO USE.

PREVENTION:

- ◆ Except for that lens saw trough, please do not use the machine for other purposes.
- ◆ Do not touch the saw on the machine as the work of the machine, so as not to be injured.
- ◆ If the power can not match (voltage and frequency), do not use the machine, the working power is (AC220V-50HZ AC110V-60HZ)
- ◆ Do not place the machine at intense heat or in direct sunlight.
- ◆ If any liquid or materials enter inside of the machine, Turn off the switch
- ◆ at once, pull out the plug, until the machine is checked by the expert, can just work again.
- ◆ Except the original parts of the machine, please do not go to assemble other parts.
- ◆ Do not let machine burden too much, do not collide any part in the machine. If do not use the machine in some time, please pull out the plug.

OPERATING INSTRUCTIONS

- (1) Open the paper carton, take out the machine wrapped up with the plastic foam.
- (2) Place the machine on the enough large and steady place.
- (3) Open to have power, start switch, make saw turning right away.
- (4) Open visor (if lens trough wide 0.8MM, please use the left saw; lens trough wide 1.0MM, using the right saw.)
- (5) The lens that will rub the good form slots on the saw, can't exert oneself to pay attention to too, in case that the electrical machinery overloads.
- (6) Please close the power and visor while sawing the good lens.



AN-1000 NOTCH-CUTTING MACHINE

