



Operation Manual

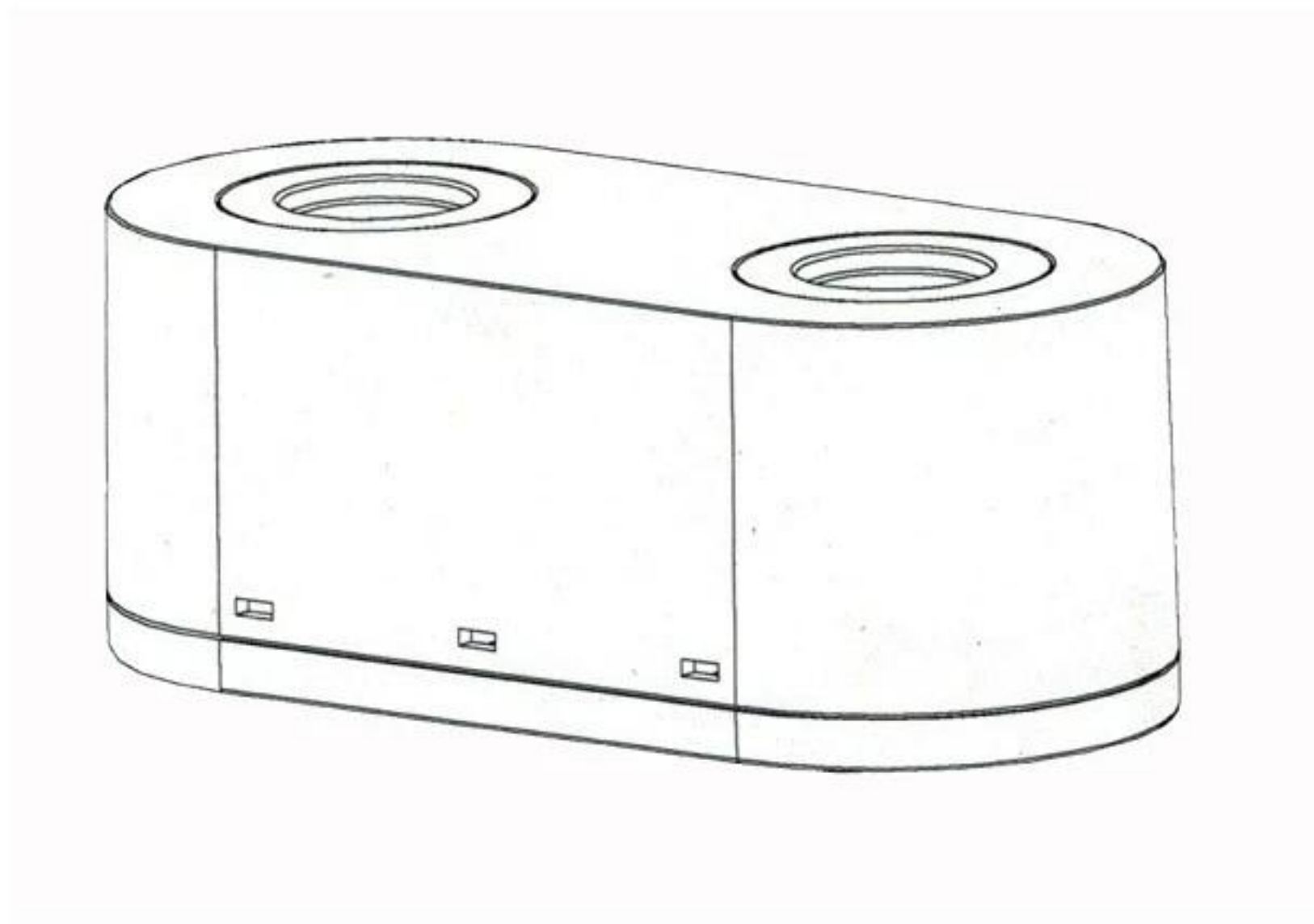

Charger User's Guide

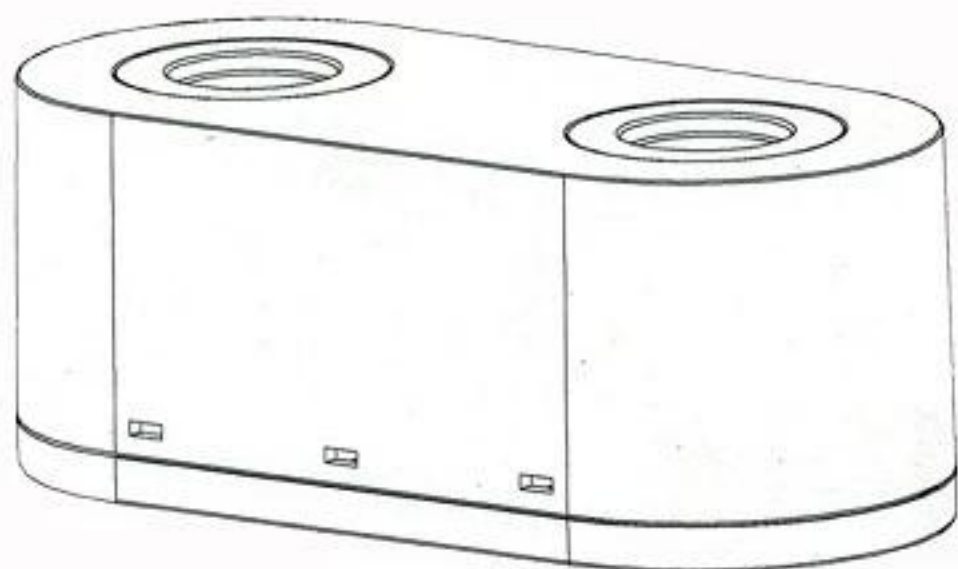

Thank you for purchasing this charger. To prevent damage to your charger or injury to yourself or to others, read the following safety precaution in their entirety before using this equipment. Keep these safety instructions where all those who use the product will read them.

Symbols:



Warnings and Cautions:



Attention. Read operating manual for cautions and instructions for use



Warning: This product must not be used in the presence of flammable gases.



Warning: This product should not be immersed in fluids.



Warning: Please ensure that the control is in the off position when the examination has been completed.



Warning: Federal law restricts this device to sale or order of a physician.

1. Purpose and features of the product

1.1 Purpose

The DT408 Charger is only intended to charge either one or two zumax rechargeable handles.

1.1 Features


- ◆ LEDs indicate power to the unit and that battery are

charging

- ◆ Automatic charging when inserting the handle unit, The battery automatically charge.
- ◆ when inserting the handle unit, the light automatically turns off.
- ◆ Switchmode Single-Cell/Two-Cell Li-ion/Li Polymer Battery Charger
- ◆ precharge-CC-CV Operation model
- ◆ circuit , reverse-voltage ,over-voltage protection.
- ◆ Low Battery Status Indicator.

2. Operating Instructions

2.1 DT408 lithium battery charger external diagram is as follows:

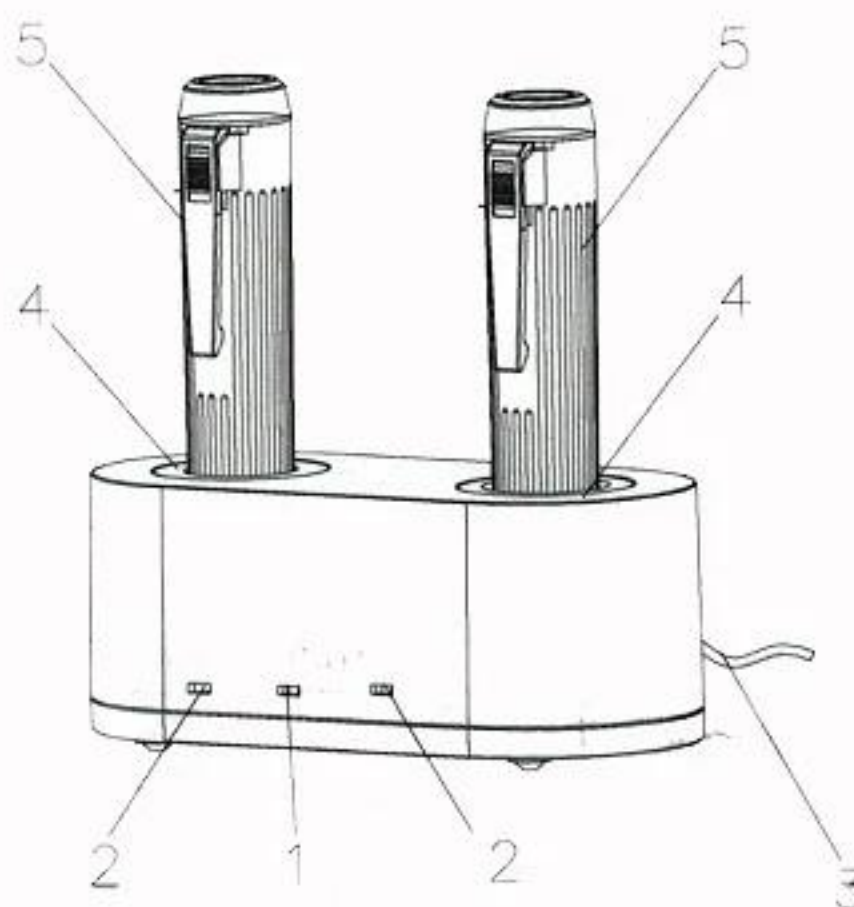

1	Power Light	The green led light indicated the charger is powered.
2	Charge Indicator Lights	the charge indicator shows yellow to indicate charging, the charging indicator light shows green indicates charging is complete
3	Power Line	Connect the power adapter
4	Charging port	Insert into either one or two Rechargeable lithium batteries
5	Rechargeable lithium batteries	Can only be inserted under DT850 rechargeable lithium battery handle, pay attention to the direction of the handle insert.

2.2. The Charger is immediately ready for use when the mains transformer plug into the mains and the charger is ready for use.. When illuminated, the center green power light indicates that the Charger is powered and can be charged

2.3 Place the rechargeable lithium battery handle into the charger as shown. the charge indicator shows yellow to indicate charging, the charging indicator light shows green indicates charging is complete.

2.4 Place the rechargeable lithium battery from the charging port .The charger to stop charging, charging indicator light turn off.

2.5 power adapter output 5V, the current 2A

3. Maintenance

3.1 Cleaning Recommendations

Wipe all external surfaces with cloth dampened with a mild detergent and water solution, a mild detergent and water solution, a solution of 70% isopropyl alcohol and water solution, or a solution of 10% Clorox and water. DO NOT immerse. Wring out the cloth to prevent excess moisture from entering the assembly.

Clean instrument windows with a cotton swab dipped in alcohol.



Caution: Be careful to prevent the solvent into the instrument.

4. Technical Specifications

Electrical Specifications		
Rated voltage	110-220VAC ~ 50Hz	
Rechargeable Lion handles model	K handle rechargeable Li-ion battery	
Power adapter	Output 5V/2A	
maximum charge time	≤ 3.5 h	
Electrical Safety Standard	IEC60601-1, IEC60601-1-2 Class II	
Transformer thermal circuit breaker	Action temperature of 130 °C	
Environment requirements		
Use Condition	Environment Temperature	+10 °C...+35 °C
	Relative Humidity	30%...75%
	Atmospheric Pressure	700 hPa...1060 hPa
Shipping and Storage	Environment Temperature	-40 °C...+55 °C
	Relative Humidity	10%...90%
	Atmospheric Pressure	500 hPa...1060 hPa

CHG-7800 Desk Charger with Power Adapter



9990 NW 14 Street | Suite 105 | Doral, FL 33172
Telephone: 888.881.1122