

User's Manual



Notification

Dear Users,

Thank you for your purchase of ET 150 Electric Table. Please take time to read our user's manual carefully before use.

This guarantees you to make full use of this unit and prolongs the operation life of this unit.

Precautions

If you have detected abnormal heat, smoke, noise or smell, immediately stop using the product.

In the event of an abnormality, turn off the power and disconnect the power plug from the power socket. Continuing to use the product may result in electric shock or fire.

Observe the instructions given below regarding the power cable:

- Be sure to use the supplied or specified power cable.
- Do not modify, forcibly bend, kink or pull the power cable.
- When disconnecting the power cable from the AC outlet, be sure to hold the cable by the plug.
Pulling the cable may cause wire breakage or short circuit, resulting in fire or electric shock.
- Do not connect or disconnect the plug of the power cable to/from the AC outlet using wet hands.
Doing so may result in electric shock.
- Do not touch the product with wet hands while the power cable is connected to the AC outlet.
Doing so may result in electric shock.
- If the product will not be used for a long period, disconnect the power cable from the power source. Leaving the cable connected to the power socket for a prolonged period will consume electricity and may result in heating.

1. Function Introduction

ET 150 Electric Table is made of imported materials and by up-to-date technology overseas. It features with aesthetic appearance of outline, satisfactory performance of manipulation, safety and credibility of operation; its load capacity can perform 55Kg to the utmost. It also has the advantages of relocation simply and conveniently, sounding slightly, lifting steadily and with large amplitude (630mm - 820mm), etc., all of which make it a matching platform for laying various desk-top apparatus such as computer, medical treatment appliances, etc.

2. Preparations

- A. Two foot-rests in package box with truckles should be installed on both sides of bottom of main machine respectively. Four M8 hexagonal socket screws should be twisted tightly by hexagonal wrench keystroke.
- B. The platform plate should be fastened by four M6 hexagonal socket screws tightly with the main machine.
- C. Connect the plugs and main machine with the power respectively (please remark this machine mate 220V/110V)

3. Accessories

M8 hexagonal socket screws	4 sets
M8 hexagonal socket screws	4 pieces
M8. M6 hexagonal wrench keys	1 piece, respectively
Drawer	1 piece
Power cable	1 piece



LUXVISION is not responsible or liable for indirect, special or consequential damages arising out of or in connection with the use or performance of the product or damages with respect to any economic loss, loss of property, loss of revenues or profits, loss of enjoyment or use, costs of removal or installation or other consequential damages of whatsoever nature. Some states do not allow the exclusion or limitation of incidental or consequential damages. Accordingly, the above limitation may not apply to you.

Every effort has been made to ensure the accuracy of this manual. However, LUXVISION, makes no warranties with respect to the documentation and disclaims any implied warranties of merchantability and fitness for a particular purpose. LUXVISION, Inc. shall not be liable for any errors or for incidental or consequential damages in connection with the furnishing, performance, or use of this manual or the examples herein. The information in this document is subject to change without notice.