

Operation Manual



Dear Users:

Thank you for purchasing our heater, which is a great honor. We strongly believe that safety precedes all the other things, which is deeply taken into consideration for this machine. But before using, please read the following items meticulously for the first time.

1. When the power button is on, the last but one light is on. It is the most commonly used temperature. Press the push button to select the temperature level. Green is cold wind. Other red lights are hot indicators.
2. After choosing the temperature level, and the red lights stop blinking, it means that the temperature has become stable at the set temperature, whose temperature tolerance is ± 3 degree.
3. The temperature will shift to cold wind after five minutes' undoing when the temp is 125 (others need 20 minutes). The temperature indicators in a row respectively are cold, 80, 90, 100, 110, 125.
4. The whole heater automatically and ultimately stops after 1 minutes blowing of cold wind.
5. Press the selecting key for 2 seconds continuously, the air will shift to cold.

Notes:

1. In order to extend the life-span, please do not turn the main switch off directly.
2. Please turn the machine off when it is blowing cold air to extend life-span
3. When the lights blink regularly from bottom to the top, it indicates that the motor or the sensor has burned out.
4. Do not use the heater in moisturized environment to avoid electricity short-circuit.



FH-6300 Frame Heater



9990 NW 14 Street | Suite 105 | Doral, FL 33172
Telephone: 888.334.4640